

Finecare™ FIA Meter III Plus

Routine Operation Instruction for Use




1. Connect the instrument with power source and turn on the device. Finecare™ FIA Meter III Plus will load on the system and enter the Log in interface. Input user name (**default: Adm**) and password (**default: 123456**) to log in. After the 1 minute self-check, the meter is ready to perform its task.



2. Check the Lot No. of ID card and test reagent before exporting the information to the instrument, ensure the Lot No. matches with each other. Insert the ID card, select "**Reagent**" and click "**Read ID card**" to export data from the ID card to Finecare™ FIA Meter III Plus.



3. Select "**Test**", a "**Start Test**" window will pop out. After choosing correct sample type and other corresponding information, click "**Confirm**" to start test.

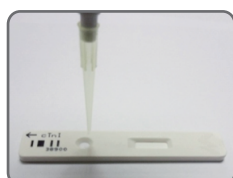

4. Collect and prepare specimens in accordance with the procedure indicated in the package insert for specific assay. Load the sample mixture to the sample well.



5. Insert the test device to Finecare™ FIA Meter III Plus. Once the test device is inserted, the test device access closes. The carousel rotates to transfer the test device into the test area. The bar code on the test device is then scanned. A time countdown will appear on the screen.



6. The instrument will automatically read and show the results as soon as the incubation time ends.



7. With the Auto Print configuration, the test results will be printed automatically. Alternatively, when the test is completed, navigate to "**Test record**". Select the needed records and click "**Print**" to print out.